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Thank you for choosing our Foot Massager Bath Gamma MD 6014. It combines the beneficial properties of water with those of a massage, exploiting the relaxing qualities and their healthy effects. The foot bath produces excellent results for treating migraines, headaches, colds and facial neuralgia. In addition, it has a beneficial effect on overtired lower limbs and varicose veins. The heated bottom keeps water at a constant temperature. A relaxing foot massage is just what you need at the end of a stressful day.

We are sure that having appraised worthily the high quality and reliability of this device you will become a regular user of the products of Gamma Trade Mark.

Before starting to use the foot massager please read the instruction manual carefully. The instruction manual offers all information you need to correctly use the device.

**ATTENTION**

Your foot massage bath is meant to play a supporting role, for use in conjunction with any treatments prescribed by your doctor.

**CONTENT**

- 1. COMPONENTS AND DESCRIPTION OF YOUR FOOT MASSAGER BATH GAMMA MD 6014**
- 2. GETTING READY FOR USAGE**
- 3. USING YOUR FOOT MASSAGER BATH**
- 4. CARE AND MAINTENANCE**
- 5. TECHNICAL SPECIFICATIONS**
- 6. WARRANTY**

## 1. COMPONENTS AND DESCRIPTION OF YOUR FOOT MASSAGER BATH GAMMA MODEL MD6014

Your foot massager bath kit includes bath, 2 platforms with massage rollers and 2 multi-node platforms, electrical cord, instruction manual, guarantee card, gift box. The picture shows the foot massager bath.

## 2. GETTING READY FOR USAGE

Before using the appliance, make sure it is intact and does not feature any visible signs of damage. If in any doubt, do not use the appliance and contact the Service Centre the address of which is stated in your guarantee card.

Make sure your hands are dry when unplugging or plugging in the appliance and when turning the function control knob.

This appliance must only be used with the voltage given on the bottom of the product.

The foot massager bath is for home use.

## ATTENTION

Using any electrical appliance entails compliance with a number of fundamental rules:

- Unplug the appliance from the mains straight after use and before commencing cleaning or

inserting/removing accessories.

- Do not unplug by tugging on either power cord or appliance.
- If an electrical appliance falls into the water, do not attempt to retrieve it; instead, unplug immediately.
- Do not use the appliance while bathing or showering.
- Never immerse the appliance in water or other liquids.
- Never leave the appliance working unattended, and never allow it to be used by children, handicapped or disabled individuals.

## 3. USING YOUR FOOT MASSAGER BATH

1. Make sure the appliance is unplugged and the switch set to "off".
2. Choose one of the interchangeable platforms supplied with the appliance: 2 platforms with massage rollers and 2 multi-node platforms.
3. Fill with hot or cold water for a relaxing or refreshing massage. Never allow the water level to exceed the maximum indicator line (MAX).  
The appliance cannot be used to heat cold water.  
If you want a massage with hot water, fill with hot water and the appliance will keep it warm. This appliance can also be used without water, for a simple foot massage.
4. Plug in and select the desired function.

Function 1: massage

The massage is produced by vibrations, which help to soothe away a whole day's stress.

This function can be used with or without water.

Do not use foaming products.

Function 2: massage + heat

This function can only be used with water.

Heat: the heated bottom keeps water at a constant temperature.

Function 3: heat

This function can only be used with water. Your foot massage is also fitted with a knotted massaging roller, placed at the centre to effectively stimulate the soles of your feet.

5. Sit down and place your feet in the spa bath.

6. After use, switch off the appliance by setting the switch to "ON".

7. Unplug the appliance from the mains and pour water out of the relevant opening.

#### **Note**

- After using the appliance for 20 minutes non-stop, it must be allowed to cool for at least 15 minutes.  
Never allow the water level to exceed the maximum indicator line.
- Take special care to ensure the air inlets on the bottom of the appliance are never clogged (e.g. hairs, fibres etc.); consequently, avoid placing the appliance on soft surfaces (e.g. bed or cushions).
- Do not use the appliance in damp rooms, or anywhere aerosols are in use or oxygen is being administered.
- Under no circumstances should you use two electrical appliances at the same time (e.g. foot massage bath and hairdryer, foot massage bath and electric razor or electric toothbrush).
- Do not stand up in the footbath: sit down before putting your feet in the water.
- The appliance cannot be used to heat cold water. If you want a massage with hot water, fill with hot water and the appliance will keep it warm.
- Anyone with health problems and people with pacemakers should consult their doctor before using the appliance.
- The appliance must always be disconnected from the mains during filling and emptying.  
Use water alone: do not add bubble bath or foaming bath salts.

#### **ATTENTION**

- If the massage is uncomfortable or causes pain either during or after treatment, consult your doctor. Under no circumstances should you use the appliance if you have wounds, skin rashes, burns or bruises on your feet.
- Do not use the appliance near eyes or in other sensitive areas.
- Do not use the appliance after taking medicine, sedatives or alcohol, as sensitivity may be reduced considerably.
- You are advised not to use the appliance on people with pacemakers, prostheses, handicaps, serious diagnosed pathologies, thrombosis, pregnant women or anyone suffering from alterations in skin sensitivity. In such cases, or if in any doubt, you are advised to consult your doctor before use.

#### **4. CARE AND MAINTENANCE**

- Clean with a damp cloth, do not use chemical or scouring agents. Dry thoroughly.
- In the event of a fault and/or irregular operation, turn the appliance off at once without tampering with it. Always contact the service centre the address of which is stated in your guarantee card.
- Keep the appliance away from sources of heat.
- This appliance must be used solely for the purpose for which it was designed and in accordance with the operating instructions. Any other use shall be considered improper and hence hazardous. The manufacturer cannot be held responsible for any damage resulting from improper or incorrect use.

#### **DISPOSAL PROCEDURE**

The symbol located on the bottom of the device indicates the separated collection of electric and electronic equipment. At the end of the useful life of the device, do not dispose it as mixed solid municipal waste, but dispose it in a specific collection centre located in your area. This procedure of separated collection of electric and electronic equipment is carried out foreseeing a common EU environmental policy aiming at safeguarding, protecting and improving the quality of environment.

#### **5. TECHNICAL SPECIFICATIONS**

Power supply: 220V  
Size: 423x355x188 mm  
Weight: 1,95 kg  
3-function switch  
Interchangeable platforms

## **6. WARRANTY**

Your Foot Massager Bath Gamma MD 6014 is warranted for 2 years from date of purchase.

Never open the appliance; in the event of any opening or tampering the warranty will automatically become invalid. The warranty does not cover either the parts subject to wear and tear or the batteries (if provided), and packing.

The warranty covers only any faults in manufacturing and does not apply to damage caused by improper handling, accidents, not following the operating instructions or self-maintained alterations made to the device.

Use only the accessories provided; the use of accessories other than the approved ones can make your warranty void.

The manufacturer shall not be liable for any damages to people, property or pets, either directly or indirectly, caused by the noncompliance with the instructions laid down in the instruction manual, in particular the warnings regarding the installation, use and maintenance of the appliance.

The warrantee is only valid upon presentation of the warrantee card which was correctly filled in and sealed.

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